


MODELS

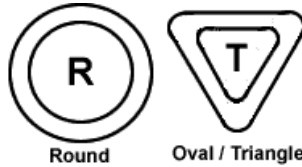

(see below for MODELS)



1 Place the bedpan on an even surface. Put your arm into the bag and place your middle finger under the strip at the bottom of the bag. Loosen the green strip.



2 Seize the edge of the bedpan (opposite the handle of the bedpan) with your thumb and your middle finger so that the glue sticks to the edge.



3 Pull the bag backwards over the bedpan, maintaining your grip of the bag and the edge of the bedpan.



4 It is important that the bag is completely shaped into the bedpan, so that the bag fits smoothly at the bottom of the bedpan.



5 Put your arm into the bag again and take hold of the green strip.



6 Pull the strip off with a sliding movement, as you press the glue against the sides and bottom of the bedpan.



7 Continue the movement all the way out on the handle.



8 The remaining part of the bag is folded around the handle, and the green strip is tied around it. After use, the contents are emptied into the toilet. Loosen the green strip.



9 Pull the bag free of the handle. Seize the handle and move your other hand in along the edge of the bedpan. Move the bag back against the handle - without causing the glue at the bottom to loosen.



10 The bag is slightly pressed, still without loosening the glue.




11 Turn the handle away from your body. Now roll the bag up backwards - against the opening of the bag - so that the glue closes the bag.



12 Now the bag can be thrown into the waste container. The green strip could be tied around the bag.

MODEL

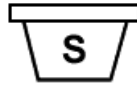

Bucket



1 Put your arm into the bag and place your middle finger under the strip at the bottom of the bag. Loosen the strip.




2 Make sure that the glue follows the pouring direction of the bucket. Now place your middle finger in the middle of the bottom of the bucket.



3 Pull the strip off with a sliding movement, as you press the glue against the bottom and side of the bucket.



4 The bag is glued entirely out over the edge of the bucket. Press the air out by forming the bag after the bucket.

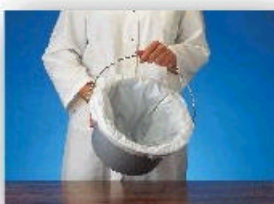

5 Fold the bag around the edge of the bucket and place it in the chair. Empty the contents out over the glue-side - at the green mark.



6 After use the bag is folded up slightly at the top - opposite the glue-side.



7 Squeeze the opening of the bag together on the edge. Then loosen the bag at the bottom.



8 Roll up the bag from the bottom, so that the glue closes the bag.



9 Now the bag can be thrown into the waste container.