




BIOSYNEX COVID-19 Ag BSS

VIROLOGY

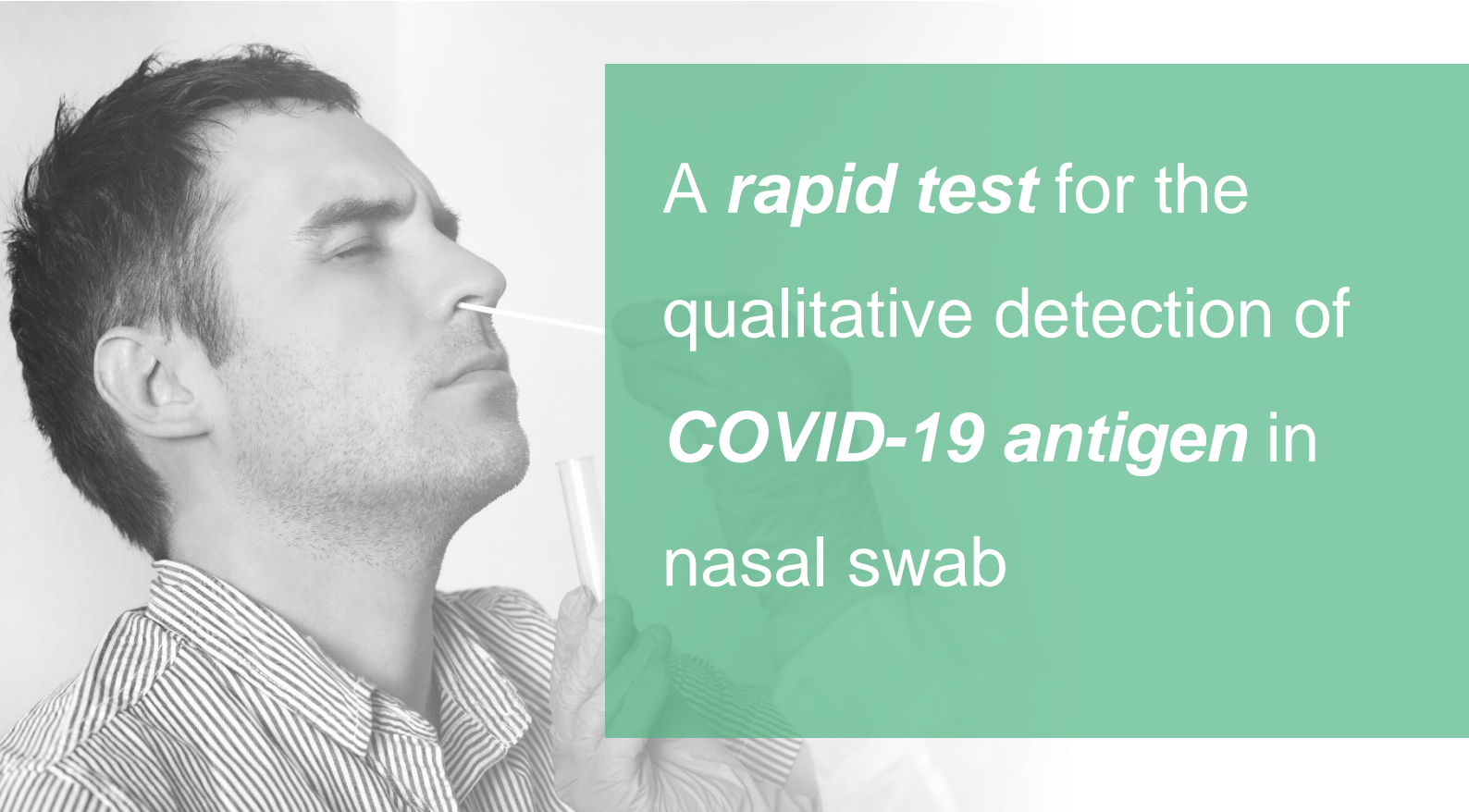

A rapid test for the qualitative detection of **COVID-19 antigen** in nasal swab



Easy to use

Using nasopharyngeal swab in only a few steps



Rapid detection of COVID 19 antigens

Results in 15 minutes



Convenient and easy to use

The kit can be stored at room temperature or refrigerated (2-30°C)






BIOSYNEX COVID-19 Ag BSS


COVID-19 is a disease caused by a novel coronavirus (CoV) SARS-CoV-2 also known as 2019-nCoV.

In order to facilitate patient management and to identify infected individuals, especially those at risk of complications, it is essential to be able to rapidly detect the presence of CoV-2-SARS.


The BIOSYNEX COVID-19 Ag BSS is a rapid chromatographic immunoassay for the qualitative detection of COVID-19 antigen, in only 15 minutes, using a nasopharyngeal swab.

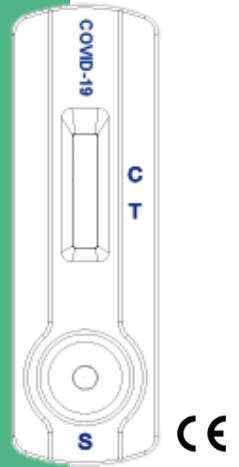


PROCEDURE

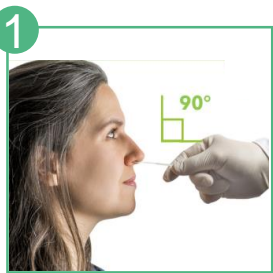


1 Insert the swab **horizontally 3 cm** inside the nasal cavity.

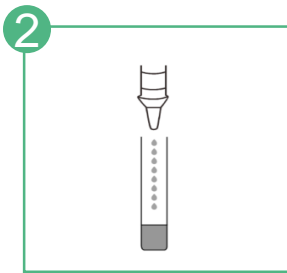


2 Add 10 drops (0,3mL) of the sample extraction buffer.

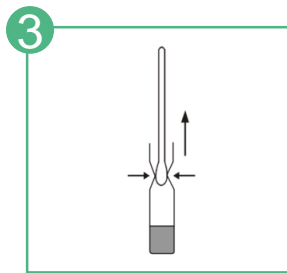

3 Discharge the swab into the extraction tube by **rotating** the swab for approximately **1 minute**.

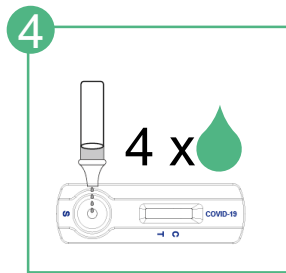


4 Screw on the cap of the extraction tube and place **4 drops** of the solution in well S.

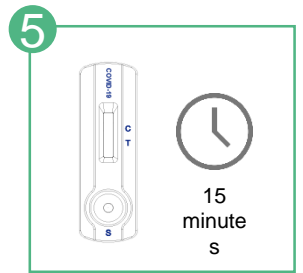

5 Read the result at **15 minutes**. Do not read after 20 minutes.

Watch the video illustrating the sampling here:
<http://youtu.be/DVJNWefmHjE>


RESULTS

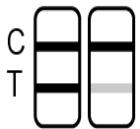

POSITIVE



NEGATIVE

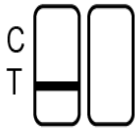

INVALID

PERFORMANCE*

- Sensitivity: >95%
- Specificity: 100%

*Compared to a PCR test. Read the instruction for use for more information.

KIT CONTENTS

BIOSYNEX COVID-19 BSS
Ref. SW40006

Contains:

- 25 test cassettes
- 25 buffer vials
- 25 nasal swabs
- Instructions for use