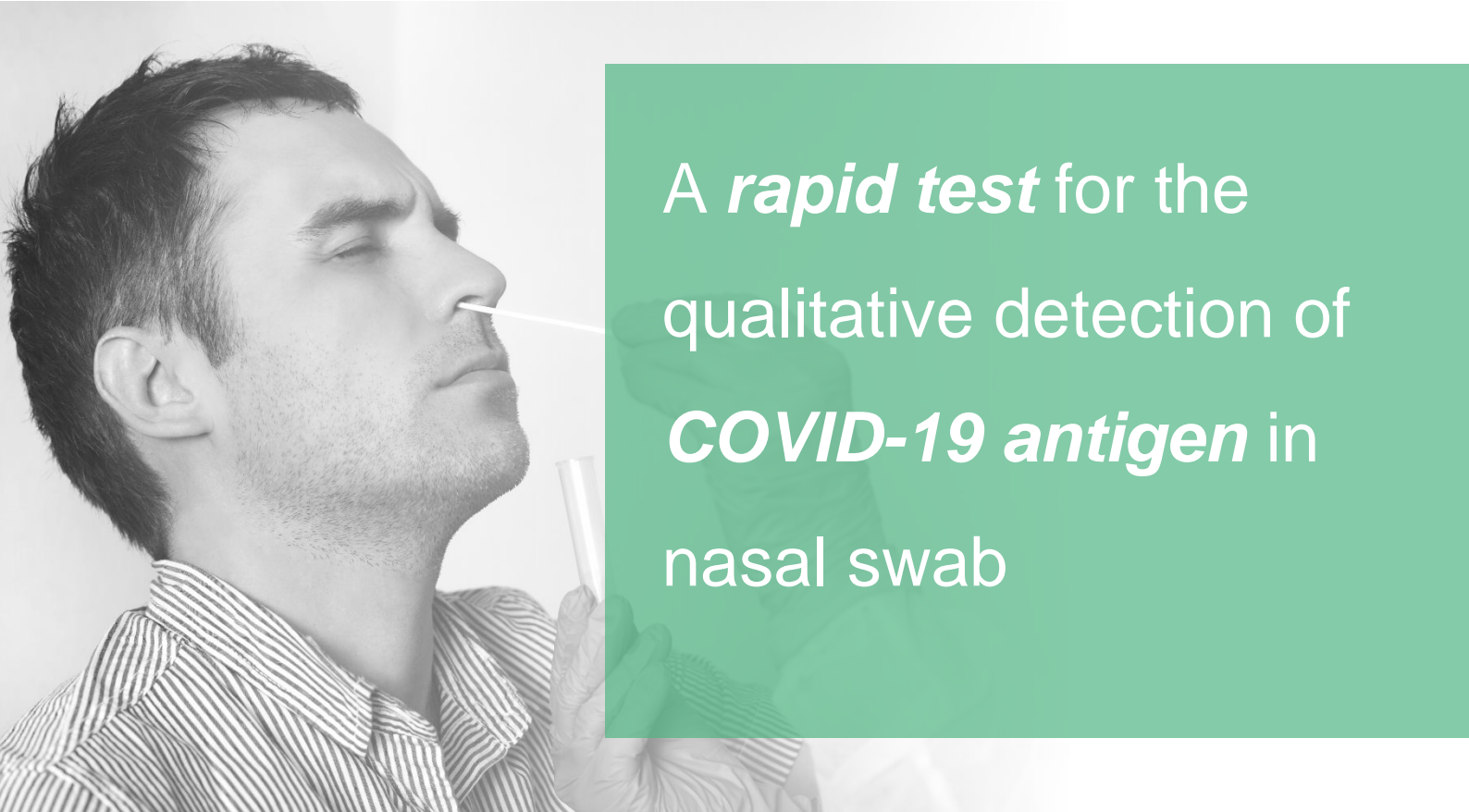




# BIOSYNEX COVID-19 Ag BSS

## VIROLOGY



*A rapid test* for the qualitative detection of **COVID-19 antigen** in nasal swab



*Easy to use*

Using nasopharyngeal swab in only a few steps



*Rapid detection of COVID 19 antigens*

Results in 15 minutes



*Convenient and easy to use*

The kit can be stored at room temperature or refrigerated (2-30°C)



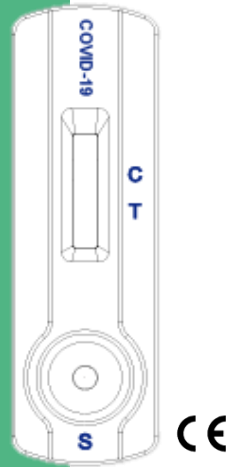


# BIOSYNEX COVID-19 Ag BSS

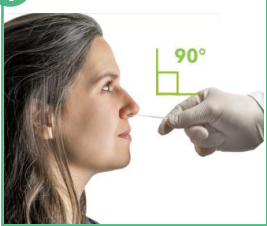
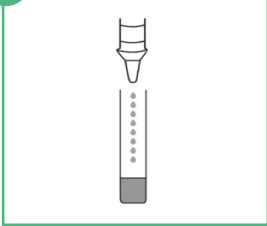
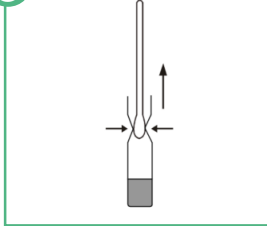
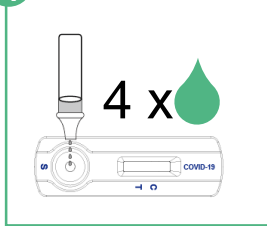
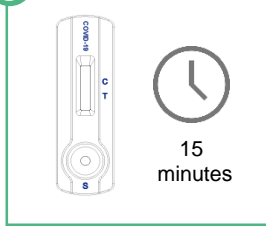
COVID-19 is a disease caused by a novel coronavirus (CoV) SARS-CoV-2 also known as 2019-nCoV.

In order to facilitate patient management and to identify infected individuals, especially those at risk of complications, it is essential to be able to rapidly detect the presence of CoV-2-SARS.

The BIOSYNEX COVID-19 Ag BSS is a rapid chromatographic immunoassay for the qualitative detection of COVID-19 antigen, in only 15 minutes, using a nasopharyngeal swab.

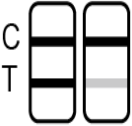
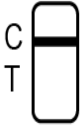
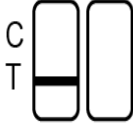


## PROCEDURE

-   
*Do a nasal cavity swab.*
-   
*Add 10 drops (0,3mL) of buffer.*
-   
*Insert the swab into the tube. See procedure cf. Instruction for use.*
-   
*Screw on the cap of the tube and place **4 drops** of the solution in well S.*
-   
*Read the result at **15 minutes**. Do not read after 20 minutes.*

Watch the video illustrating the sampling here:  
<http://youtu.be/DVJNWefmHjE>

## RESULTS

		
POSITIVE	NEGATIVE	INVALID

## PERFORMANCE\*

- Sensitivity: 96%
- Specificity: 100%

\*Compared to a PCR test. Read the instruction for use for more information.

## KIT CONTENTS

**BIOSYNEX COVID-19 BSS**  
Ref. SW40006

Contains:

- 25 Test cassettes
- 2 Buffer
- 25 Sterile Swabs
- 25 Extraction tubes
- 1 Workstation
- 1 Package insert